



**PLEASE NOTE THE MEDICATION CUP IS A REPLACEABLE PART  
ITS LIFE IS VERY DEPENDENT ON THE DRUG NEBULISED AND HOW WELL IT IS MAINTAINED**



This chart is meant to be used as a guide for you and your veterinarian. Some of the medications listed are not available in all countries.  
It is recommended to always consult your veterinarian for advice regarding diagnosis and the treatment protocol for your horse.

### Antibiotics / Antifungal

Drug	Trade Name	Drug Format	Dilute with 0.9% Saline	Med Cup Colour Code	Special Med Cup Colour Code For Shorter Delivery Time
Cefquinome	Cobactan®	Cobactan® 4.5% Injectable Solution	No	GREY	GREEN
Ceftiofur Sodium	Excenel®	Excenel® 4G Injectable Solution	No	GREY	GREEN
Gentamicin	Gentaject	Gentaject 10% Injectable Solution	Yes 1:1	GREY	GREEN
Amikacin	Amikavet	Amikavet Injectable Solution	Yes 1:1	GREY	GREEN
Marbofloxacin	Marbocyl	Injectable Solution	Yes 1:1	GREY	GREEN
Nystatin	R-Nystatine	Oral Solution	Yes 1:1		GREEN

### Bronchodilators

Drug	Trade Name	Drug Format	Dilute with 0.9% Saline	Med Cup Colour Code
Ipratropium	Atrovent®	Atrovent® 250 UDVS Nebuliser Solution	No	GREY
Salbutamol (Albuterol)	Ventolin™	Ventolin™ 2.5 Nebuliser Solution	No	GREY
Clenbuterol	Ventipulmin®	Ventipulmin® 30 Injectible Solution	Yes 1:1	GREY

### Corticosteroids

Drug	Trade Name	Drug Format	Dilute with 0.9% Saline	Med Cup Colour Code	Special Med Cup Colour Code For Shorter Delivery Time
Dexamethasone	Dexameth, Chem-Pharm Dexadreson, MSD Animal Health DexaJect SP, Henry Schein Ilium Dexapent, Troy Laboratories	Dexameth: 2mg/ml Injectable Solution Dexadreson: 2mg/ml Injectable Solution DexaJect SP: 4mg/ml Injectable Solution Ilium Dexapent: 5mg/ml Injectable Solution	Yes 1:1	GREY	GREEN
Dexamethasone (high viscosity / contains polyethylene glycol 400)	Azium®, VetOne® VEDCO, Duphacort®, Dexamethasone 5 by Vétquinol	Injectable Solutions	Yes 2:1 (Saline:Dex)		GREEN
Fluticasone Propionate	Flixotide™	Flixotide™ 2mg/2ml Nebuliser Suspension	No		GREY
Budesonide	Pulmicort®	Pulmicort® 0.5 Nebuliser Solution	Yes 1:1		GREY

### Mucolytics

Drug	Trade Name	Drug Format	Dilute with 0.9% Saline	Med Cup Colour Code
Acetylcysteine	Parvolex® ACC Injekt by HEXAL Equimucin®	Parvolex® 2g Solution for Infusion Acetylcysteine 300mg/3ml Injection Solution Acetylcysteine 200mg/ml Injection Solution	Yes 1:1	GREEN
0.9% Saline Solution	0.9% Saline Solution	B.Braun NaCl 0.9%	No	GREY
Bromhexine	Bisolvon	Oral Solution	Yes 1:1	GREY
Emser Salt	Emser Salt	Nebuliser Salt Solution	No	GREY

### Other Therapies

Active Ingredients	Trade Name	Format	Dilute with 0.9% Saline	Med Cup Colour Code
Mixture of essential oils solubilised for Inhalation	Immuno-San	Nebuliser Solution	No	GREEN
Chelated silver and distilled water	SilvaPlex™ & EquiSilver™	Oral Solution	No	GREY
Hyaluronic acid, eucalyptus essential oil in isotonic solution	Hyalusol®	Nebuliser Solution	No	GREEN
Grindelia, water and essential oils	Balsamic Air	Oral Solution	Yes 1:1	GREEN
Natural hydro-soluble extract of curcuma longa	Nebulin®	Powder + NaCl	No	BLUE

## Med Cup Colour Code Guide



- The Grey STANDARD label med cup is the standard cup that will nebulise a wide range of medications
- A Green FAST label cup is specially designed for use with difficult to nebulise high viscosity medications
- A Blue SLOW label cup is specially designed for use with the natural therapy Nebulin®

### Important Information:

The med cup & drug combination choice can have a big impact on the delivery time and aerosol droplet size.

- A **GREY** labelled cup used with high viscosity oily type drugs can lead to slow delivery or failure to nebulise.
- A **GREEN** labelled cup used with low viscosity watery type drugs can lead to very fast delivery and the generation of big aerosol droplets leading to pools of medication wasted in the bottom of the mask chamber.
- A **BLUE** labelled cup used with low viscosity watery type drugs will nebulise but the delivery time will be slow. It is not suitable for high viscosity oily type drugs.
- If the drug you wish to nebulise is not listed on the drug table then it is best to test a sample of the drug with your cup first to check if it nebulises ok and in an acceptable time before delivering the treatment.
- If you are unsure about what med cup to use it is best to contact your local distributor for advice.
- Check out our website for updates to this information or email [technicalsupport@nortev.com](mailto:technicalsupport@nortev.com) for advice.
- Please note the following are **NOT** to be nebulised with Flexineb; *RESPRADIL, VETADINE, BROMINE, MILTON / BLEACH, COLLOIDAL SILVER, DE-IONISED WATER, IONISED WATER*